

# SleepStrip®



Disposable Sleep Apnea Screener

With ~80% of OSA patients still undiagnosed - effective, low-cost screening is good news.



## SleepStrip Advantages

- ▶ At-home study, in the patient's natural sleeping environment
- ▶ Easy to use, disposable, patient-applied test
- ▶ Low cost
- ▶ Accurate, good correlation with sleep lab results
- ▶ Small, lightweight, can be sent by mail
- ▶ Instant, permanent results, no additional equipment necessary

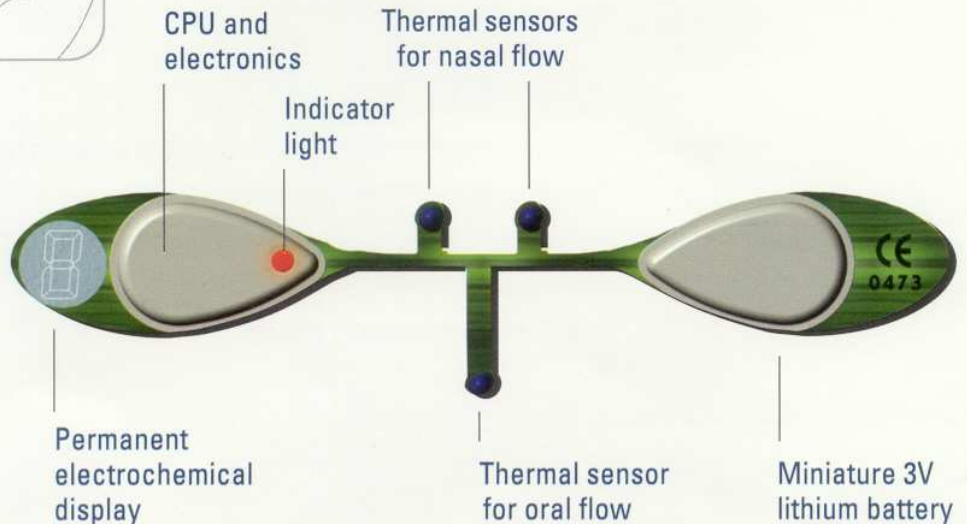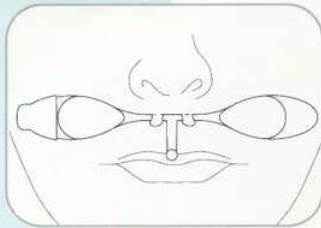

US and international patents  
granted and pending

**CARE**  
*Medical Limited*  
Cardiac, Anaesthetic, Respiratory Equipment  
8G PIERMARK DRIVE, N.H.I.E. ALBANY  
TELEPHONE: (09) 915-0800, FAX: (09) 915-0801  
EMAIL: sales@caremed.co.nz



# SleepStrip®

Disposable Sleep Apnea Screener

## Can you detect sleep apnea?

Just look at SleepStrip to read the results, and see for yourself:

- No sleep apnea - less than 15 events per hour
- Mild sleep apnea - 15 to 24 events per hour
- Moderate sleep apnea - 25 to 39 events per hour
- Severe sleep apnea - 40 or more events per hour
- Error - study too short or other problems

SleepStrip single-use sensors provide the easiest, most advanced and cost-effective method for accurate screening of SAS. SleepStrip's accuracy has been validated by leading international decision makers.

SleepStrip is a complete all-in-one system. There's no need for additional equipment or components. Study results are read directly off SleepStrip's built-in display the morning after the study.

## How SleepStrip Works

Three miniature flow sensors monitor the patient's respiration all through the night. The integrated CPU analyzes respiration patterns to detect and count each apnea and hypopnea event as it occurs. The morning after the study the SAS severity score is calculated based on the patient's AHI and permanently displayed on the built-in electrochemical display.

### SLP Ltd.

18 Hazfira Street,  
Tel-Aviv 67779, Israel  
Tel: +972-3-5371281  
Fax: +972-3-5371282  
E-mail: slp.ltd@slp.co.il

Our distributor:

**CARE**  
*Medical Limited*  
Cardiac, Anaesthetic, Respiratory Equipment  
8G PIERMARK DRIVE, N.H.I.E, ALBANY  
TELEPHONE: (09) 915-0800, FAX: (09) 915-0801  
EMAIL: sales@caremed.co.nz